

TECHNOSERVE



A Leading Name in POCT & Diagnostics

POCT Devices & Bio Markers From Global Leaders


Comprehensive Quantitative POCT markers for

- ▶ Cardiovascular Diseases Bio-Markers
- ▶ Infectious Diseases Bio-Markers
- ▶ Drug of Abuse Test (DAT)
- ▶ Dengue Tests (IgG/IgM & NS1)
- ▶ Covid-19 IgG/IgM Antibody Test
- ▶ Nicotine (Cotinine) Test
- ▶ Cancer Bio-Markers
- ▶ Malaria Pf/PAN Ag

Nanochecker 710(V1)



Automatic Immunoassay Analyzer

- ▶ Large 10.1" Colored Touch Screen
- ▶ A true point of care Quantitative testing solution
- ▶ Principle of Immunofluorescence
- ▶ Proprietary Image Photometry Technology
- ▶ No calibration or maintenance required
- ▶ Self-diagnostic and internal quality control checks
- ▶ In-built Thermal Printer
- ▶ A fully portable system works on battery/AC power
- ▶ QC Controls, External Controls Available
- ▶ Stores more than 10,000 test results
- ▶ Weight: <1.7 Kg
- ▶ Support: All necessary Upgrades & Repairs through Remote Desktop Assistance

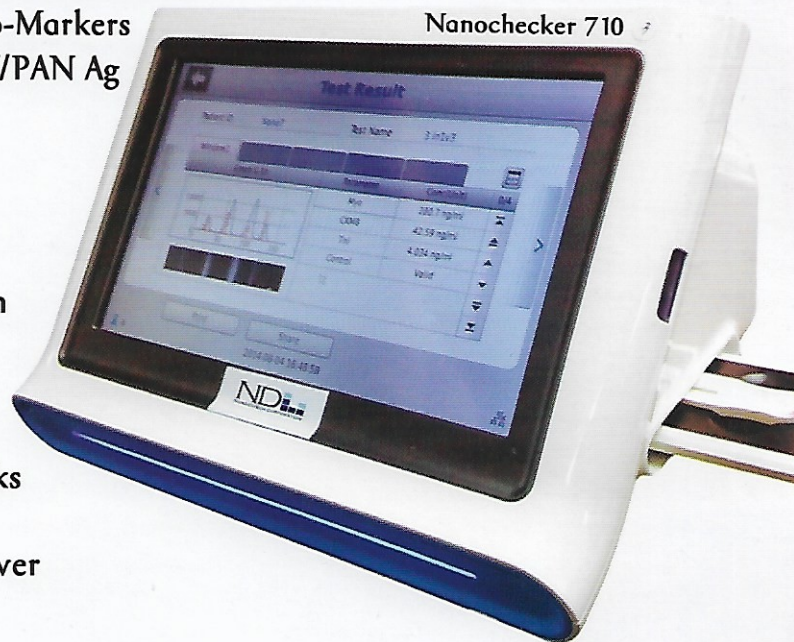

Nano-check Test Kit

- ▶ Automatic Detection of Qualitative & Quantitative Point-of-Care-Test Result
- ▶ Built-in Control in Each Panel
- ▶ Quick & Accurate Fast Results within 10-15 min
- ▶ Shelf Life upto 24 Months
- ▶ Storage : Room Temp.(2-30°C)
- ▶ Sample : 80µl W.B./P/S/Urine & 10µl with Finger Prick.
- ▶ Pack Size : 20/25 Test

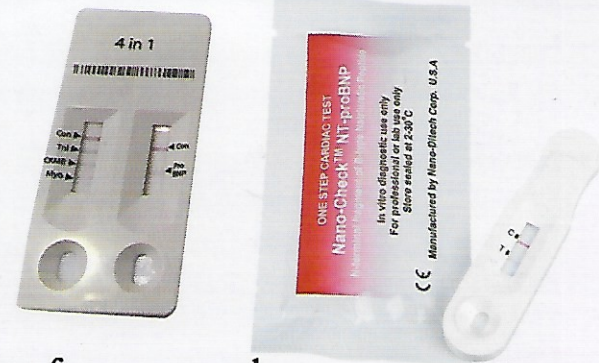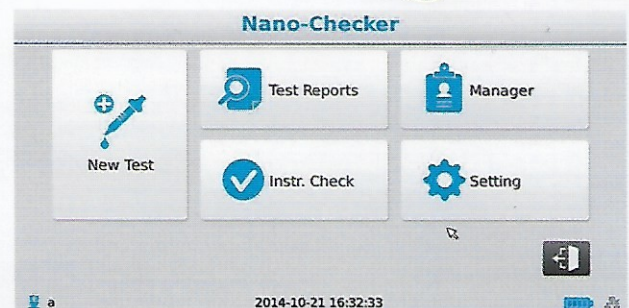

Connectivity

- ▶ Results transmitted to computers, via LIS/LAN to save effort for paperwork
- ▶ Storage, Data Export through Pen drive/email & Direct A-4 Size Printout from a Local Printer

Exclusive Indian Partner for



Certifications:



POCT Measured Parameters

Quantitative & Qualitative

- Nano-Check™ AMI-I cTnI (Cardiac Troponin I)**
As an aid in the diagnosis of Acute Myocardial (AMI)
- Nano-Check™ NT-proBNP**
One Step Test for CHF (Congestive Heart Failure)
- Nano-Check™ AMI-III (3-in-1 Test) CK-MB/cTnI/Myo**
As an aid in the diagnosis of myocardial damage
- Nano-Check™ AMI-IV (4-in-1 Test) CK-MB/cTnI/Myo/NT-proBNP**
As an aid in the diagnosis of myocardial damage & CHF
- Nano-Check™ D - Dimer (Finger-Prick)**
For the exclusion of Deep-Vein thrombosis (DVT) & Pulmonary Embolism (PE)
- Nano-Check™ PCT (WB/Finger-Prick)**
Detect of Procalcitonin (PCT) as an aid in the early diagnosis of severe bacterial infection leading to multi organ failure
- Nano-Check™ CRP (C Reactive Protein) (Finger-Prick)**
For the assessment of bacterial infection and diagnosis of inflammation
- Nano-Check™ hs-CRP (High Sensitive - C Reactive Protein) (Finger-Prick)**
For the assessment of cardiovascular disease and diagnosis of inflammation
- Nano-Check™ PSA (Prostrate Specific Antigen Cancer Marker)**
For the assessment of presence of prostrate specific antigen
- Nano-Check™ DAT (Drug of Abuse Test - 10T/8T/6T)**
For the presence of various drugs present in the body (urine)
- Nano-Check™ Nicotine (Cotinine) Test**
For the presence of cotinine in the body (urine)
- Nano-Check™ Dengue IgG/IgM Test**
For the presence of Dengue Virus- IgG/IgM in human whole blood/plasma/serum
- Nano-Check™ Dengue NS1 Test**
For the presence of Dengue Virus- NS1 Ag in human whole blood/plasma/serum
- Nano-Check™ Malaria Pf/PAN Ag Test**
For the presence of Malaria Pf/PAN Ag (PLDH & HRP-2) in human whole blood/plasma/serum
- Nano-Check™ COVID-19 IgG/IgM Antibody Test**
One-step, Rapid, Immuno-Chromatographics Assay for the qualitative detection of human IgG/IgM antibodies against SARS-CoV-2 in human whole blood, serum and plasma.



Test Result - PCT


Test Information		Patient Information	
Test Name	Nano-Check PCT	ID	sahibchand
Cat. No.	PCT	Name	
Lot Number	WT610100467	Birth	
Expiration Date	2016-10	Gender	

- Test Result Information -

Applied Sample Information

The number of window	1
Description of test	PCT Control
The number of line	2
Reading time	15:00
Sample Type	Whole Blood
Operator ID	a
Test Date	2016-02-10

- Result -

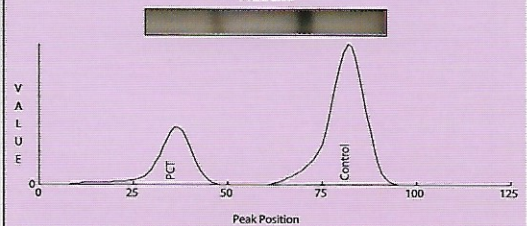

Parameter	Result(Unit)	Ref. Range
PCT	3.33 ng/ml	<0.25
Control	Valid	Control

Test Result - DAT - 10 T


Test Information		Patient Information	
Test Name	Nano-Check DAT 10	ID	8
Cat. No.	DAT 10	Name	
Lot Number	CESB1A	Birth	
Expiration Date	2016-5	Gender	

- Test Result Information -


Applied Sample Information

The number of window	2
Description of test	PCP, MET, COC, OPI, THC, Control / BZO, TCA, MDN, AMP, BAR, Control
The number of line	6 / 6
Reading time	15:00
Sample Type	Urine
Operator ID	a
Test Date	2015-10-13

- Result -

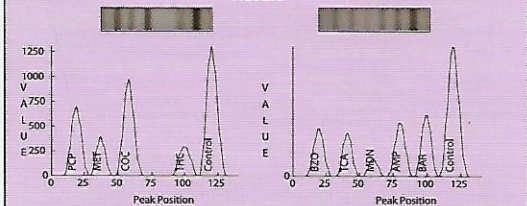

Parameter	Result	Score(Unit)	Parameter	Result	Score(Unit)
PCP	Negative	789.99	BZO	Negative	415.92
MET	Negative	479.33	TCA	Negative	392.25
COC	Negative	1089.41	MDN	Negative	165.73
OPI	Positive	0.00	AMP	Negative	472.38
THC	Negative	332.62	BAR	Negative	555.31
Control	Valid	Control	Control	Valid	Control

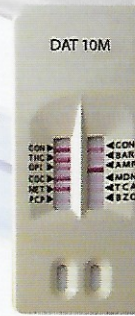

Manufactured By: **NDI** 259 Prospect Plains Road, Building K, Cranbury, NJ 08512 (USA)

Marketed in India Exclusively by :
TECHNOSERVE ENTERPRISES
Technology with Uncompromised Service



Distributed By :

